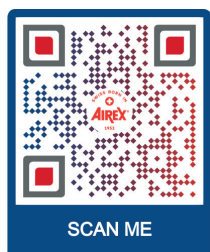
## Bruksanvisning S34\_Balance-produkter

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|                                                                                     |                                                                                                                                                      |                                                                                     |                                                                                                                                                                                                |
|-------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|    | <b>Fabrikant / Manufacturer</b><br>Indicates the medical device manufacturer                                                                         |    | <b>Temperaturbegrensninger / Temperature limit</b><br>Indicates the temperature limits to which the medical device can be safely exposed                                                       |
|    | <b>Autorisert person / Authorized Representative in the EC</b><br>Indicates the authorized representative in the European Community / European Union |    | <b>Skal ikke brukes hvis pakningen er skadet / Do not use if package is damaged</b><br>Indicates a medical device that shouldn't be used if the package has been damaged                       |
|    | <b>Fabrikasjonsdato / Date of manufacture</b><br>Indicates the date when the medical device was manufactured                                         |    | <b>Følg bruksanvisningen / Consult instructions for use</b><br>Indicates the need for the user to consult the instructions for use                                                             |
|  | <b>Batch code / Batch code</b><br>Indicates the manufacturer's batch code so that the batch or lot can be identified                                 |  | <b>Advarsel/ Caution</b><br>To indicate that caution is necessary when operating the device or control close to where the symbol is placed                                                     |
|  | <b>Artikkel/ Catalog number</b><br>Indicates the manufacturer's catalog number so that the medical device can be identified                          |  | <b>Medisinsk / Medical device</b><br>Indicates the item is a medical device                                                                                                                    |
|  | <b>Serienummer / Serial number</b><br>Indicates the manufacturer's serial number so that a specific medical device can be identified                 |  | <b>Oversettelse / Translation</b><br>To identify that the original medical device information has undergone a translation                                                                      |
|  | <b>Oppbevares vekk fra sollys / Keep away from sunlight</b><br>Indicates a medical device that needs protection from light sources                   |  | <b>I denne retningen oppover / This way up</b>                                                                                                                                                 |
|  | <b>Hold deg tørr / Keep dry</b><br>Indicates a medical device that needs protection from moisture                                                    |  | <b>CE-merking / CE marking</b><br>'CE marking of conformity' or 'CE marking' means a marking by which a manufacturer indicates that a device is in conformity with the applicable requirements |
|  | <b>Unik produktidentifikator / Unique device identifier</b><br>Indicates a carrier that contains unique device identifier information                |  | <b>Importør / Importer</b><br>To indicate the entity importing the medical device into the locale                                                                                              |
|                                                                                     |                                                                                                                                                      |  | <b>Representant / Distributor</b><br>To indicate the entity distributing the medical device into the locale                                                                                    |

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With the super soft products, great importance is attached to functionality. In order to achieve the softness a special manufacturing process is necessary. Certain deviations from perfection may occur with these products, such as occasionally small surface bubbles, small deviations in dimensional tolerance or slight discoloration. However, this does not affect the function in any way.

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Dirt of all kinds is preferably cleaned with a soft brush and handwarm soapy water. Stains and similar impurities must not be treated with petrol or stain remover. If disinfection is necessary, it can be carried out with commercially available disinfectants such as Merfen, Descogen or Chromium. Sterilization with superheated steam (autoclave sterilization) or hot water is not permitted.

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The closed-cell AIREX® soft foams do not absorb any water and have a permanent load-bearing capacity in water.

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ming or rescue aid!

With these extremely robust and hard-wearing materials, however, a pressure loss occurs in the individual cells when pressure is applied over a long period of time (similar to a balloon). This results in pressure marks which, however, generally disappear again slowly.

AIREX® soft foams are equipped with an antibacterial agent and therefore offer permanently effective protection against bacterial and fungal attack\*.

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\* The antimicrobial Sanitized® hygiene function integrated into our balance trainer protects the material from odours, bacteria and mold. Contains a biocidal product with the active ingredient folpet. Concentration: 0.1 – 0.2%.

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AIREX® Balance-products are best stored lying flat. To avoid pressure points, objects should not be left on the material. A continuous pressure load of more than for more than approx. 12 hours should be avoided. Bent storage (e.g. over a clothesline) leaves visible pressure marks, which will not disappear completely. Prolonged storage in water and prolonged exposure to direct sunlight shorten the lifetime of the products, as in the course of time surface embrittlement and stiffening may occur. To extend the service life we recommend that you rinse the products with clear water after use, shake off the water and dry in the air (because of pressure points over radiators, etc.) dry in the air.

Under no circumstances should you leave your Balance-pad on an oiled parquet floor for any length of time.

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Think of the environment when disposing of packaging material and product and please enquire at your local administration about the possibilities of environmentally friendly disposal via appropriate collection points.

AIREX® balance-trainers are made of soft foam based on polyvinyl chloride cellular materials (PVC) and are expanded with a nitrogen-releasing chemical blowing agent. The material does not contain phthalates.

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We provide a legal guarantee for AIREX® Balance pads unless otherwise stated below.

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Claims due to damage of the AIREX® balance trainer which are based on improper use are excluded. Claims for damages - irrespective for whatever legal reason, in particular due to violation of obligations arising from the and from tortious acts – are excluded. This shall not apply insofar as liability is mandatory by law, in particular:

- in cases of intent or gross negligence;
- in the event of injury to life, body or health;
- under the Product Liability Act or

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• in the event of any other breach of material contractual obligations;  
In this case, the claim for compensation claim for damages is limited to the typically occurring damage.

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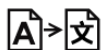
| Country               | Business Partner             | Address                                                                                    | Phone             | Email                                    |
|-----------------------|------------------------------|--------------------------------------------------------------------------------------------|-------------------|------------------------------------------|
| <b>Latvia</b>         | G Kolizejs Ltd.              | Brīvības gatve 439<br>LV- 1024 Rīga                                                        | +37 16 799 40 44  | info@gfitness.lv                         |
| <b>Latvia</b>         | SIA BALTMEDIKA               | Sencu prospekts 51,<br>Garkalnes distr.<br>LV- 2164 Baltezers                              | +37 16 799 02 61  | info@baltmedika.lv                       |
| <b>Lithuania</b>      | TEIDA UAB                    | Puodziu g. 4-1<br>LT- 44288 Kaunas                                                         | +37 03 732 35 76  | info@teida.lt                            |
| <b>Netherlands</b>    | CONVET BV                    | Bergveste 20<br>NL- 3992 DE Houten                                                         | +31 30 634 17 50  | informatie@convet.nl                     |
| <b>Norway</b>         | AlfaCare AS                  | Heddalsveien 11<br>NO- 3674 Notodden                                                       | +47 35 02 95 95   | post@alfacare.no                         |
| <b>Norway</b>         | FYSIOPARTNER AS              | Ryensvingen 15<br>NO- 0680 Oslo                                                            | +47 23 05 11 60   | salg@fysiopartner.no                     |
| <b>Norway</b>         | KLUBBEN AS                   | Færdervegen 49<br>NO- 2270 Flisa                                                           | +47 62 95 06 10   | info@klubben.no                          |
| <b>Poland</b>         | Meden-Inmed Sp. z o.o.       | Wenedów 2<br>PL- 75-847 Koszalin                                                           | +48 94 347 10 40  | meden@meden.com.pl                       |
| <b>Poland</b>         | Klubben Polska Sp. z o.o.    | Ciele, Torunska 12<br>PL- 86-005 Białe Błota                                               | +48 52 345 66 03  | info@klubben.pl                          |
| <b>Romania</b>        | Mens Mentis                  | Strada Mihai Pavel 23,<br>RO- 41210 Oradea                                                 | +40 35 944 44 43  | office@fitness-shop.ro                   |
| <b>Russia</b>         | MFitness                     | Nakhimovskiy prospect,<br>56, RU- 117335 Moscow                                            | +74 95 974 12 34  | info@mfitness.ru                         |
| <b>South Korea</b>    | Ducwoo PT Ltd.               | SHINWON ACHIM DOSI<br>MAHIN B1, 30 Dosan-<br>daero 85-gil, Gangnam-<br>gu, KR- 06012 Seoul | +821 02 220 15 07 | ducwoo@ducwoo.com                        |
| <b>Spain</b>          | RehabMedic                   | C/ Berenguer de Palou<br>86, local 1, ES- 08027<br>Barcelona                               | +34 93 352 76 50  | contacto@rehabmedic.com                  |
| <b>Sweden</b>         | MEDEMA PHYSIO AB             | Torshamnsgatan 35,<br>SE- 16440 Kista                                                      | +46 8 404 12 00   | info@medema.se                           |
| <b>United Kingdom</b> | Physical Co.                 | The Works, Wycombe<br>Road, HP14 3RR Stoken-<br>church, Bucks                              | +44 149 476 92 22 | sales@physicalcompany.co.uk              |
| <b>United Kingdom</b> | Jordan Fitness               | 56 Oldmedow Road,<br>Hardwick Industrial Es-<br>tate, PE30 4JJ Kings<br>Lynn Norfolk       | +44 155 376 32 85 | sales@jordanfitness.co.uk                |
| <b>United Kingdom</b> | Performance Health           | Nunn Brook Road, Huth-<br>waite, NG17 2 HU Sutton-<br>in-Ashfield, Nottingham-<br>shire    | +44 344 873 00 35 | UKRetail-<br>Sales@performancehealth.com |
| <b>USA</b>            | Fabrication Enterprises, Inc | PO Box 1500<br>10602 White Plains, NY<br>USA                                               | +1 914 345 93 00  | info@fab-ent.com                         |

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Airex AG
x AG
**3A Composites GmbH**