
AIREX®

Bruksanvisning S50_Treningsmatter

AIREX®
CADEMY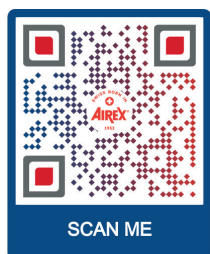

AIREX®
YOGA & PILATES

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The following symbols are used on labels, packaging or instructions for use of AIREX® products

	Fabrikant / Manufacturer Indicates the medical device manufacturer		Temperaturbegrensninger / Temperature limit Indicates the temperature limits to which the medical device can be safely exposed
	Autorisert person / Authorized Representative in the EC Indicates the authorized representative in the European Community / European Union		Skal ikke brukes hvis pakningen er skadet / Do not use if package is damaged Indicates a medical device that shouldn't be used if the package has been damaged
	Fabrikasjonsdato / Date of manufacture Indicates the date when the medical device was manufactured		Følg bruksanvisningen / Consult instructions for use Indicates the need for the user to consult the instructions for use
	Batch kode / Batch code Indicates the manufacturer's batch code so that the batch or lot can be identified		Advarsel / Caution To indicate that caution is necessary when operating the device or control close to where the symbol is placed
	Artikkel / Catalog number Indicates the manufacturer's catalog number so that the medical device can be identified		Medisinsk / Medical device Indicates the item is a medical device
	Serienummer / Serial number Indicates the manufacturer's serial number so that a specific medical device can be identified		Oversettelse / Translation To identify that the original medical device information has undergone a translation
	Oppbevares vekk fra sollys / Keep away from sunlight Indicates a medical device that needs protection from light sources		I denne retningen oppover / This way up
	Hold deg tørr / Keep dry Indicates a medical device that needs protection from moisture		CE-merking / CE marking 'CE marking of conformity' or 'CE marking' means a marking by which a manufacturer indicates that a device is in conformity with the applicable requirements
	Unik produktidentifikator / Unique device identifier Indicates a carrier that contains unique device identifier information		Importør / Importer To indicate the entity importing the medical device into the locale
			Representant / Distributor To indicate the entity distributing the medical device into the locale

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Great importance is attached to functionality in the super-soft products. A special manufacturing process is necessary to achieve good cushioning. Some deviations from perfection may occur with these products, such as occasional small surface bubbles, small deviations in the dimensional tolerance or slight discolorations. However, this does not affect the function in any way.

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To enjoy your gymnastics mat for a long time, we strongly recommend cleaning the mat at regular intervals, especially if it is used in extremely warm and humid environments (e.g., public, or private baths, thermal baths, wellness areas). This will prevent any changes in the surface structure and increase the service life. For cleaning, use a soft sponge or cloth with lukewarm soapy water. Please do not use scouring agents or scouring sponges. Avoid cleaning with petrol or other organic solvents. After cleaning, allow the mat to air dry thoroughly before storing.

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The closed-cell AIREX® soft foams do not absorb water and have a permanent load-bearing capacity in water. **Attention: Do not use as a swimming or rescue aid!** The Sanitized® finish effectively protects the gymnastics mat against unpleasant odours and microbial infestation. It preserves the material, improves hygiene and increases comfort.* If disinfection is nevertheless necessary for special reasons, please use a commercially available water-based disinfectant. Disinfection or sterilisation with superheated steam is not permitted.

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* The antimicrobial Sanitized® hygiene function integrated in our mats protects the material against odours, bacteria as well as mould. Contains a biocidal product with the active ingredient Folpet. Concentration: 0.1 - 0.2 %.

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Single mats can be rolled up without further ado. Make sure that the grooves are facing outwards. Storage on our special holders is ideal for storing several mats, provided your mats are equipped with AIREX® eyelets. Please always hang the mats on both eyelets. If you lose eyelets, you can replace them without any problem. Replacement eyelets can be obtained from our service department. (see Contact).

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Think of the environment when disposing of packaging material and product and please enquire at your local administration about the possibilities of environmentally friendly disposal via appropriate collection points. AIREX® gymnastic mats are made of flexible foam based on polyvinyl chloride cellular materials (PVC) and are expanded with a nitrogenreleasing chemical blowing agent. The material does not contain any phthalates.

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We provide the prescribed legal warranty for AIREX® gymnastic mats, unless otherwise specified below.

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Claims for damage to the AIREX® gymnastics mat due to improper use are excluded.

Claims for damages - irrespective of the legal grounds, in particular due to breach of obligations arising from the contractual obligation and from unauthorized actions - are excluded. This shall not apply insofar as liability is mandatory by law, in particular:

- in cases of intent or gross negligence;
- in cases of injury to life, body or health;

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
Country	Business Partner	Address	Phone	Email
Latvia	G Kolizejs Ltd.	Brīvības gatve 439 LV- 1024 Riga	+37 16 799 40 44	info@gfitness.lv
Latvia	SIA BALTMEDIKA	Sencu prospekts 51, Garkalnes distr. LV- 2164 Baltezers	+37 16 799 02 61	info@baltmedika.lv
Lithuania	TEIDA UAB	Puodziu g. 4-1 LT- 44288 Kaunas	+37 03 732 35 76	info@teida.lt
Netherlands	CONVET BV	Bergveste 20 NL- 3992 DE Houten	+31 30 634 17 50	informatie@convet.nl
Norway	AlfaCare AS	Heddalsveien 11 NO- 3674 Notodden	+47 35 02 95 95	post@alfacare.no
Norway	FYSIOPARTNER AS	Ryensvingen 15 NO- 0680 Oslo	+47 23 05 11 60	salg@fysiopartner.no
Norway	KLUBBEN AS	Færdervegen 49 NO- 2270 Flisa	+47 62 95 06 10	info@klubben.no
Poland	Meden-Inmed Sp. z o.o.	Wenedów 2 PL- 75-847 Koszalin	+48 94 347 10 40	meden@meden.com.pl
Poland	Klubben Polska Sp. z o.o.	Ciele, Torunska 12 PL- 86-005 Biale Blota	+48 52 345 66 03	info@klubben.pl
Romania	Mens Mentis	Strada Mihai Pavel 23, RO- 41210 Oradea	+40 35 944 44 43	office@fitness-shop.ro
Russia	MFitness	Nakhimovskiy prospect, 56, RU- 117335 Moscow	+74 95 974 12 34	info@mfitness.ru
South Korea	Ducwoo PT Ltd.	SHINWON ACHIM DOSI MAHIN B1, 30 Dosan- daero 85-gil, Gangnam- gu, KR- 06012 Seoul	+821 02 220 15 07	ducwoo@ducwoo.com
Spain	RehabMedic	C/ Berenguer de Palou 86, local 1, ES- 08027 Barcelona	+34 93 352 76 50	contacto@rehabmedic.com
Sweden	MEDEMA PHYSIO AB	Torshamnsgatan 35, SE- 16440 Kista	+46 8 404 12 00	info@medema.se
United Kingdom	Physical Co.	The Works, Wycombe Road, HP14 3RR Stoken- church, Bucks	+44 149 476 92 22	sales@physicalcompany.co.uk
United Kingdom	Jordan Fitness	56 Oldmedow Road, Hardwick Industrial Es- tate, PE30 4JJ Kings Lynn Norfolk	+44 155 376 32 85	sales@jordanfitness.co.uk
United Kingdom	Performance Health	Nunn Brook Road, Huth- waite, NG17 2 HU Sutton- in-Ashfield, Nottingham- shire	+44 344 873 00 35	UKRetail- Sales@performancehealth. com
USA	Fabrication Enterprises, Inc.	PO Box 1500 10602 White Plains, NY USA	+1 914 345 93 00	info@fab-ent.com

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Airex AG
3A Composites GmbH