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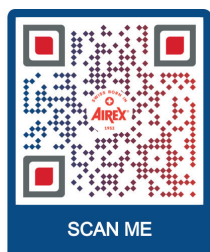
	Tillverkare / Valmistaja Indicates the medical device manufacturer		Temperaturbegränsningar / Lämpötilarajoitukset Indicates the temperature limits to which the medical device can be safely exposed
	Valtuutettu henkilö / Valtuutettu henkilö Indicates the authorized representative in the European Community / European Union		Använd inte om förpackningen är skadad / Älä käytä, jos pakkaus on vaurioitunut Indicates a medical device that shouldn't be used if the package has been damaged
	Tillverkningsdatum / Valmistuspäivämäärä Indicates the date when the medical device was manufactured		Följ instruktionerna / Noudattaa ohjeita Indicates the need for the user to consult the instructions for use
	Batchkod / Eränumero Indicates the manufacturer's batch code so that the batch or lot can be identified		Varning / Varoitus To indicate that caution is necessary when operating the device or control close to where the symbol is placed
	Artikelnummer / Tuotenumero Indicates the manufacturer's catalog number so that the medical device can be identified		Medicinsk teknik / Lääketieteellinen teknologia Indicates the item is a medical device
	Serienummer / Sarjanumero Indicates the manufacturer's serial number so that a specific medical device can be identified		Översättning / käännös To identify that the original medical device information has undergone a translation
	Håll borta från solljus / Säilytettävä poissa auringonvalolta Indicates a medical device that needs protection from light sources		Upp åt det här hållet / Ylös tähän suuntaan
	Håll torrt / Pidä kuivana Indicates a medical device that needs protection from moisture		CE märke / CE merkki 'CE marking of conformity' or 'CE marking' means a marking by which a manufacturer indicates that a device is in conformity with the applicable requirements
	Unik produktidentifierare / Ainulaatuinen tuotetunniste Indicates a carrier that contains unique device identifier information		Importör / Maahantuoja To indicate the entity importing the medical device into the locale
			Distributör / Jakelija To indicate the entity distributing the medical device into the locale

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Great importance is placed on functionality with the super soft products. In order to achieve good cushioning, a special manufacturing process is necessary. These products may experience certain deviations from perfection, such as occasional small surface bubbles, small deviations in dimensional tolerance or slight discoloration. However, this does not affect the function in any way.

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All types of dirt are preferably cleaned with a soft brush and lukewarm soapy water. Stains and similar contamination must not be treated with gasoline or stain removers.

If disinfection is necessary, this can be done with commercially available disinfectants such as NOVADEST FRESH S. Sterilization with hot steam (autoclave sterilization) or hot water is not permitted.

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The closed-cell AIREX® soft foams do not absorb water and have permanent load-bearing capacity in the water. **Attention:** Do not use as a swimming or rescue aid! However, with these extremely robust and hard-wearing materials, long-term pressure loads result in

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a pressure loss in the individual cells (similar to balloons). This creates pressure points, which generally disappear slowly. AIREX[®] soft foams are equipped with an antibacterial agent and therefore offer permanently effective protection against bacterial and fungal infestation.*

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* The antimicrobial Sanitized[®] hygiene function integrated into our balance trainer protects the material from odors, bacteria and mold. Contains a biocidal product with the active ingredient Folpet. Concentration: 0.1 – 0.2%.

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AIREX[®] Balance products are best stored lying flat. To avoid pressure points, objects should not be left lying on the material all the time. A continuous pressure load of more than approx. 12 hours should be avoided. Bent storage (e.g. on a clothesline) leaves visible pressure marks, which no longer completely disappear. Prolonged storage in water and long-term direct sunlight shorten the lifespan of the products, as the surface can become brittle and stiffen over time. To extend the service life, we recommend rinsing the products with clear water after use, shaking off the water and letting them air dry (due to pressure points, not over radiators, etc.).

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When disposing of packaging material and products, think about the environment and please ask your local administration about the options for environmentally friendly disposal via appropriate collection points.

AIREX[®] balance trainers are made of soft foam based on polyvinyl chloride cell materials (PVC) and are expanded with a nitrogen-releasing chemical blowing agent. The material does not contain any phthalates.

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We provide the required statutory warranty for the AIREX[®] balance trainers, unless otherwise specified below.

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Claims for damage to the AIREX[®] balance trainer resulting from improper use are excluded.

Claims for damages are excluded – regardless of the legal basis, in particular due to breach of obligations arising from the contractual relationship and unlawful acts. This does not apply if liability is required by law, in particular:

- in cases of intent or gross negligence;
- in the event of injury to life, body or health;
- according to the product liability law or

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• in the event of any other breach of
essential contractual obligations;
In this case, the claim for compen-
sation for damage is limited to the
damage that typically occurs.

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Airex AG
Industrie Nord 26
CH-5643 Sins
Schweiz
Tel.: +41 (0)41 789 66 66
Email: info@my-airex.com
www.my-airex.com

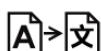
Airex AG
Industrie Nord 26
CH-5643 Sins
Sveitsi
Tel.: +41 (0)41 789 66 66
Email: info@my-airex.com
www.my-airex.com

Airex AG
Industrie Nord 26
CH-5643 Sins
Switzerland
Tel.: +41 (0)41 789 66 66
Email: info@my-airex.com
www.my-airex.com

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Country	Business Partner	Address	Phone	Email
Latvia	G Kolizejs Ltd.	Brīvības gatve 439 LV- 1024 Rīga	+37 16 799 40 44	info@gfitness.lv
Latvia	SIA BALTMEDIKA	Sencu prospekts 51, Garkalnes distr. LV- 2164 Baltezers	+37 16 799 02 61	info@baltmedika.lv
Lithuania	TEIDA UAB	Puodziu g. 4-1 LT- 44288 Kaunas	+37 03 732 35 76	info@teida.lt
Netherlands	CONVET BV	Bergveste 20 NL- 3992 DE Houten	+31 30 634 17 50	informatie@convet.nl
Norway	AlfaCare AS	Heddalsveien 11 NO- 3674 Notodden	+47 35 02 95 95	post@alfacare.no
Norway	FYSIOPARTNER AS	Ryensvingen 15 NO- 0680 Oslo	+47 23 05 11 60	salg@fysiopartner.no
Norway	KLUBBEN AS	Færdervegen 49 NO- 2270 Flisa	+47 62 95 06 10	info@klubben.no
Poland	Meden-Inmed Sp. z o.o.	Wenedów 2 PL- 75-847 Koszalin	+48 94 347 10 40	meden@meden.com.pl
Poland	Klubben Polska Sp. z o.o.	Ciele, Torunska 12 PL- 86-005 Białe Błota	+48 52 345 66 03	info@klubben.pl
Romania	Mens Mentis	Strada Mihai Pavel 23, RO- 41210 Oradea	+40 35 944 44 43	office@fitness-shop.ro
Russia	MFitness	Nakhimovskiy prospect, 56, RU- 117335 Moscow	+74 95 974 12 34	info@mfitness.ru
South Korea	Ducwoo PT Ltd.	SHINWON ACHIM DOSI MAHIN B1, 30 Dosan- daero 85-gil, Gangnam- gu, KR- 06012 Seoul	+821 02 220 15 07	ducwoo@ducwoo.com
Spain	RehabMedic	C/ Berenguer de Palou 86, local 1, ES- 08027 Barcelona	+34 93 352 76 50	contacto@rehabmedic.com
Sweden	MEDEMA PHYSIO AB	Torshamnsgatan 35, SE- 16440 Kista	+46 8 404 12 00	info@medema.se
United Kingdom	Physical Co.	The Works, Wycombe Road, HP14 3RR Stoken- church, Bucks	+44 149 476 92 22	sales@physicalcompany.co.uk
United Kingdom	Jordan Fitness	56 Oldmedow Road, Hardwick Industrial Es- tate, PE30 4JJ Kings Lynn Norfolk	+44 155 376 32 85	sales@jordanfitness.co.uk
United Kingdom	Performance Health	Nunn Brook Road, Huth- waite, NG17 2 HU Sutton- in-Ashfield, Nottingham- shire	+44 344 873 00 35	UKRetail- Sales@performancehealth.com
USA	Fabrication Enterprises, Inc	PO Box 1500 10602 White Plains, NY USA	+1 914 345 93 00	info@fab-ent.com

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3A Composites GmbH